

HOVERJACK[®]

Battery Transport System

**Part Number
HJBSC-200**



Made in USA



513 S. Clewell Street
Bethlehem, PA 18015
800-471-2776
www.HoverMatt.com



Cart Features



Left Side—AC Power Panel



Right Side - DC Power Panel



Battery Transport System Instructions

Installation

Place AIR200G or AIR400G into Air Supply Brackets located on the bottom of the cart. Align on/off switch of the Air Supply with the left front wheel. Secure hose into the Air Hose Mounts on the bottom center and right side of the cart. Wrap cord around the lower left side Power Cord Hook. Remove battery charger from the top compartment of the battery and place into Battery Charger Bracket. Wrap electrical and charging cord around upper Power Cord Hooks. Place folded HoverJack® in top basket.

Operating Instructions

To operate, remove the DC plug from the DC Power Socket and turn on the AC Outlets. Plug an AIR200G or AIR400G into one of the AC Outlets on the battery cart. Turn on the air supply and proceed with the HoverJack or HoverMatt operation as trained.

Charging the Battery

To recharge, plug the AC Charger into a standard 115-volt AC outlet and the DC plug (cigarette lighter end) into the DC Power Socket (cigarette lighter receptacle) located on the DC Power Panel. Turn off the AC switches on the battery unit and turn on the Battery Charger. The recharge Indicator on the AC Charger will change from amber to green when charging is complete (up to 15 hours). It is safe to leave the AC charger connected indefinitely. The battery's charge level may be seen by pressing the Battery Status Indicator Switch on the AC Power Panel.

Notes:

- Always keep the battery plugged in and charging when not in use to preserve battery life.
- The battery cart is not for use with the HTAIR1200. Use only HoverTech's AIR200G or AIR400G with the battery cart.



- See battery & charger owner's guides for complete safety instructions.