



HOVERJACK®

Air Patient Lift

User Manual

Visit www.HoverMatt.com for other languages

TABLE OF CONTENTS

Symbol Reference	2
Intended Use and Precautions.....	2
Part Identification - Hoverjack® Air Patient Lift	3
Part Identification - HT-Air® Air Supply.....	4
HT-Air® Keypad Functions	4
Air200G/Air400G Air Supplies.....	4
Instructions for Use	5
Product Specifications/Required Accessories.....	5
Cleaning and Preventive Maintenance	6
Returns and Repairs.....	6

Symbol Reference



DECLARATION OF CONFORMITY
TO MEDICAL DEVICE DIRECTIVE

Intended Use and Precautions

INTENDED USE

In the event of a patient fall, the HoverJack® Air Patient Lift is used to lift the patient in a supine position from the floor to bed or stretcher height, utilizing the HoverTech Air Supply to inflate each of the four chambers.

INDICATIONS

- Patients unable to assist in their own vertical lift, such as after a fall.
- Patients whose weight or girth poses a potential health risk for the caregivers responsible for lifting or moving said patients.

CONTRAINDICATIONS

- Patients who are experiencing thoracic, cervical or lumbar fractures that are deemed unstable should not use the HoverJack unless a clinical decision has been made by your facility.

INTENDED CARE SETTINGS

- Hospitals, long term or extended care facilities.

PRECAUTIONS

- Do not secure patient safety straps until HoverJack is completely inflated to desired height.
- Make sure patient safety straps are secured before moving.
- Never use patient safety straps to pull the HoverJack, as they may tear.
- Move the HoverJack using the transport handles located along the top perimeter.
- When moving a patient on the inflated HoverJack, use caution and move slowly.
- Additional caregivers are recommended when moving a patient over 350 lbs/158.757 kg.

- Never attempt to move a patient on an un-inflated HoverJack.
- Never leave a patient unattended on an inflated HoverJack.
- Use this product only for its intended purpose as described in this manual.
- Only use attachments and/or accessories that are authorized by HoverTech International.


WARNINGS/CAUTION

- For safety, always use a minimum of three caregivers while using the HoverJack.
- One caregiver should be on either side of patient during entire inflation.
- Third caregiver should manage device inflation.
- Reference product-specific user manuals for additional operating instructions.


PRECAUTIONS – HOVERTECH INTERNATIONAL AIR SUPPLY

- Not for use in the presence of flammable anesthetics or in a hyperbaric chamber or oxygen tent.
- Route the power cord in a manner to ensure freedom from hazard.
- Avoid blocking the air intakes of the HoverTech International Air Supply.
- CAUTION: Avoid electric shock. Do not open HoverTech International Air Supply.

Part Identification - HoverJack® Air Patient Lift

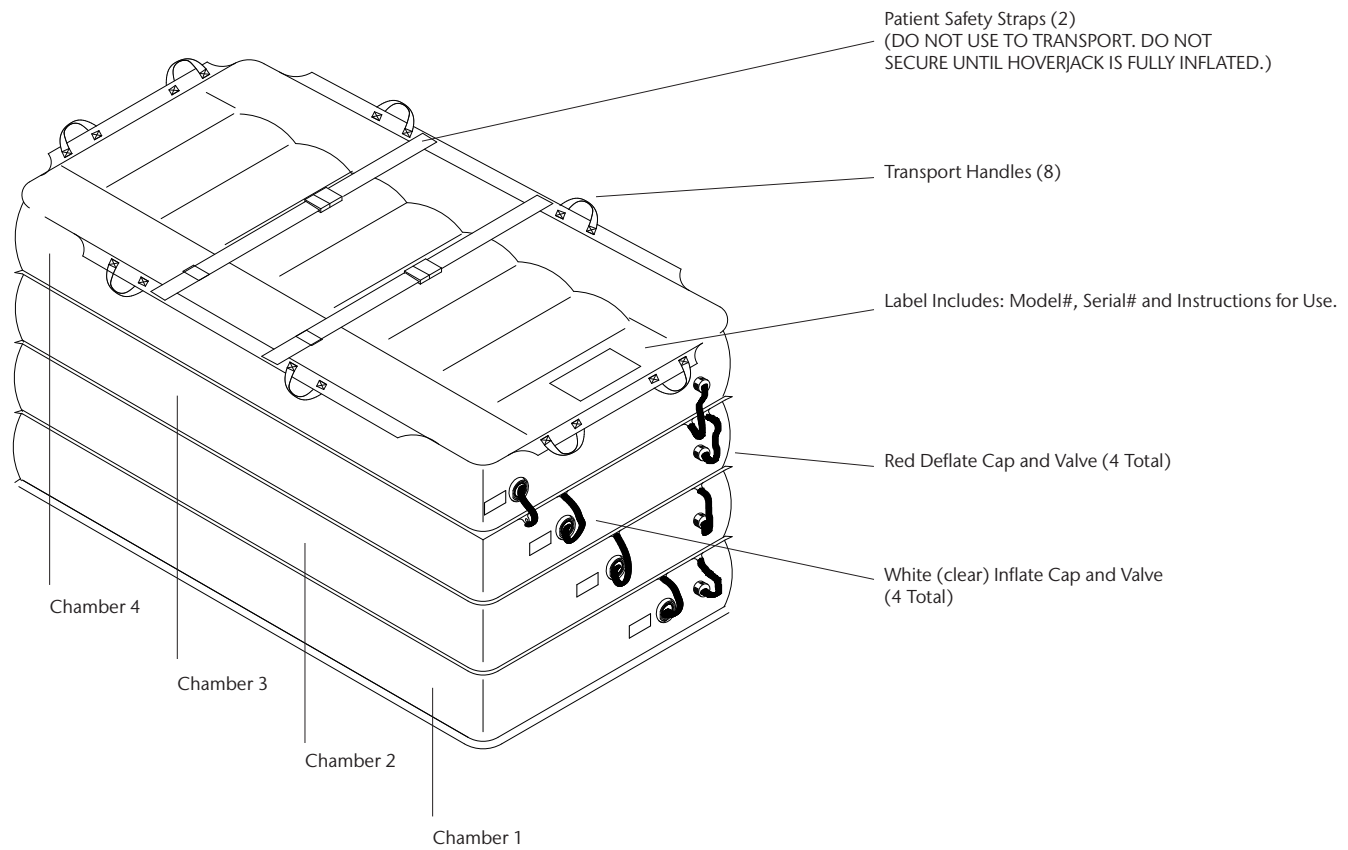


Part Identification – HT-Air® Air Supply

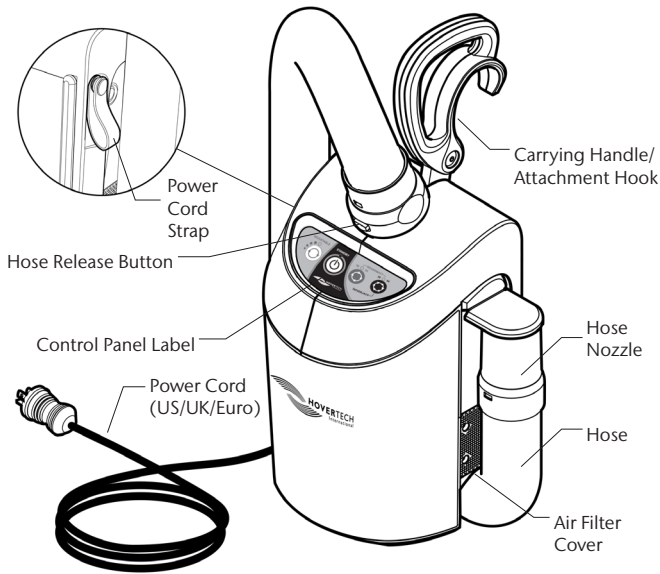

WARNING: The HT-Air is not compatible with DC power supplies.
The HT-Air is not for use with the Hoverjack Battery Cart.

HT-Air® Keypad Functions

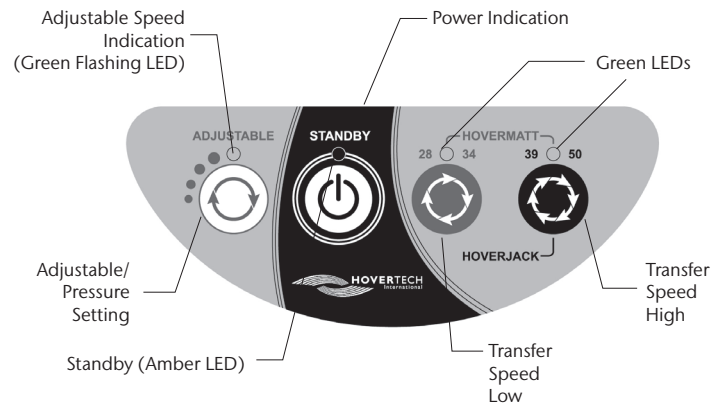

ADJUSTABLE: For use with HoverTech air-assisted positioning devices. There are four different settings. Each press of the button increases the air pressure and rate of inflation. The Green Flashing LED will indicate the inflation speed by the number of flashes (i.e. two flashes equals the second inflation speed).

All of the settings in the ADJUSTABLE range are substantially lower than the HoverMatt and Hoverjack settings. The ADJUSTABLE function is not to be used for transferring.

The ADJUSTABLE setting is a safety feature that can be used to ensure the patient is centered on HoverTech air-assisted devices and to gradually accustom a patient who is timid or in pain to both the sound and functionality of the inflated devices.



STANDBY: Used to stop inflation/air flow (Amber LED indicates STANDBY mode).



HOVERMATT 28/34: For use with 28" & 34" HoverMatts and HoverSlings.



HOVERMATT 39/50 & HOVERJACK: For use with 39" & 50" HoverMatts and HoverSlings and 32" & 39" Hoverjacks.

Air200G/Air400G Air Supplies

If using HoverTech's Air200G or Air400G Air Supplies, press the grey button on the top of the canister to initiate air flow. Press the button again to stop air flow.

Instructions for Use

1. Place the HoverJack on the floor next to the patient, making sure the chamber with valve #4 is on the top and the chamber with valve #1 is against the floor.
2. Make certain that all four red-capped deflation valves are capped tightly.
3. Log roll patient onto the deflated HoverJack, and position patient with feet at the valve end where indicated.
4. The patient can be placed on top of the HoverJack using the HoverMatt® Air Transfer System (see HoverMatt manual for instructions). If the HoverMatt is used, make certain that the HoverMatt and patient are properly centered on the HoverJack. Always deflate the HoverMatt prior to inflating the HoverJack. Ensure a caregiver is positioned at each side of patient at all times.
5. Plug HoverTech International Air Supply power cord into an electrical outlet.
6. Caregiver at foot end turns on HoverTech International Air Supply to initiate air flow.
7. To begin inflation, hold hose against inlet valve #1 of HoverJack. (valve closest to the floor)

8. When fully inflated, remove hose. Valve will automatically close, keeping the chamber inflated.

CHAMBERS MUST BE FULLY INFLATED TO ENSURE STABILITY.

9. Using the same process: move to valve #2, valve #3 and valve #4 in exact succession, or until the HoverJack reaches desired height.
10. Secure patient safety straps.
11. Turn off HoverTech International Air Supply, and attach valve caps, if desired.
12. Move HoverJack to desired location.
13. If transferring patient from the HoverJack onto an adjacent surface, unbuckle patient safety straps.
14. If it is necessary to lower the patient, release air by opening the uppermost red deflate valve #4. When chamber #4 is fully deflated, move in succession downward to fully deflate HoverJack.

WARNING: DO NOT RELEASE MULTIPLE CHAMBERS AT ONCE.

Product Specifications/Required Accessories

Material:	Topside material: Nylon oxford Underside material: Teflon® infused polyester
Construction:	RF-Welded
Width:	32" (81cm) or 39" (99 cm)
Length:	72" (183 cm) or 77" (196 cm)
Height:	30" (76 cm) Inflated [each chamber 7 1/2" (19 cm)]

LATEX FREE

Model #: HJ3202T

Model #: HJ3902T

Weight limit: 1200 lbs (544 kg)

REQUIRED ACCESSORY:

Model #: HTAIR1200 (North American Version) – 120V~, 60 Hz, 10A

Model #: HTAIR2300 (European Version) – 230V~, 50 Hz, 6A

Model #: HTAIR1000 (Japanese Version) – 100V~, 50/60 Hz, 12.5A

Model #: HTAIR2356 (Korean Version) – 230V~, 50/60 Hz, 6A

Model #: Air200G (800 W) (North American Version) – 120V~, 60 Hz, 10A

Model #: Air400G (1100 W) (North American Version) – 120V~, 60 Hz, 10A

Cleaning & Maintenance

OPTION 1 - HOVERJACK CLEANING INSTRUCTIONS

In between patient uses, the HoverJack should be wiped down with a cleaning solution used by your hospital for medical equipment disinfection. A 10:1 bleach solution (10 parts water: one part bleach) or disinfectant wipes can also be used. NOTE: Cleaning with bleach solution may discolor fabric.

First remove any visible soil, then clean the area according to the cleaning product manufacturer's recommended dwell time and level of saturation. A gentle scrub brush can also be used on the affected area, if necessary, to help penetrate the HoverJack material.

OPTION 2 - WASHING THE HOVERJACK

1. Ensure HoverJack is completely inflated and all red and clear caps are on tightly.
2. Using soap and water (maximum water temperature 160° F) and a scrub brush, gently scrub the soiled HoverJack to remove all bioburden.
3. Fully rinse the HoverJack - NOTE: Allow as little water as possible to hit the end with the caps on it.
4. Stand HoverJack on end, valves down to drain as much water as possible. Leave in this position to allow to dry completely before storing.

WARNING: DO NOT LAUNDRY THE HOVERJACK IN A WASHING MACHINE.

Returns and Repairs

All products being returned to HoverTech International (HTI) must have a Returned Goods Authorization (RGA) number issued by the company. Please call (800) 471-2776 and ask for a member of the RGA Team who will issue you an RGA number. Any product returned without an RGA number will cause a delay in the repair time.

Returned products should be sent to:

HoverTech International
Attn: RGA # _____
4482 Innovation Way
Allentown, PA 18109

PREVENTIVE MAINTENANCE

The HoverJack should be periodically inspected to ensure the following:

- All deflation valves are equipped with a red cap.
- The red caps are intact.
- All patient safety straps are attached.
- All buckles and zippers (if applicable) are intact and operational.
- Transfer handles are all attached.
- Inlet valves are all self-sealing with no evident leakage.
- There are no punctures or tears.

AIR SUPPLY CLEANING AND MAINTENANCE

See air supply manual for reference.

NOTE: CHECK YOUR LOCAL/STATE/FEDERAL/INTERNATIONAL GUIDELINES BEFORE DISPOSAL.



4482 Innovation Way
Allentown, PA 18109

800.471.2776
Fax 610.694.9601

www.HoverMatt.com
Info@HoverMatt.com